

EICKWARM H₂O

WARMING UNIT FOR SMALL ANIMALS

QUICK GUIDE




Art. No. 648020

TELEPHONE +49 7461 96 580 0
www.eickemeyer.com


veterinary technology for life
Eickemeyer

1. DEVICE

1.1 Front

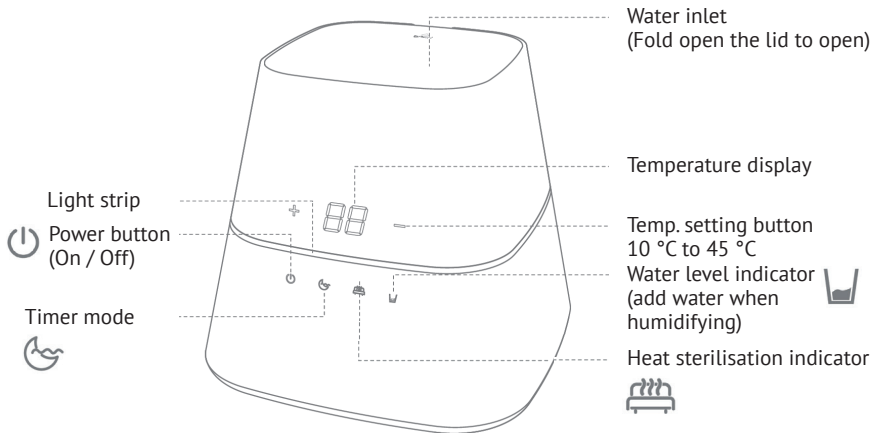


1.2 Back

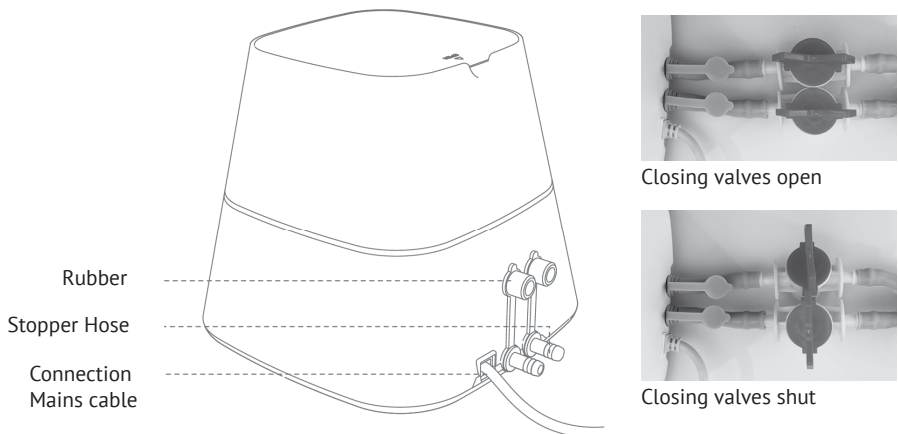


2. PREPARATION FOR USE

Step 1: Place the device on a flat surface

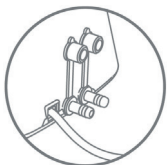

- Ensure that there is a distance **more than 5 cm** between the device and the heating mat.
- The device should be at the same height or slightly higher than the heating mat.
- Do not place the device on an uneven surface (automatic switch-off may be triggered).

Step 2: Connect the shut-off valves and the hose to the device

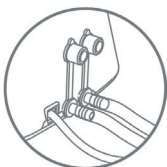


- Connect the two shut-off valves to the device.
- Connect the hoses of the heating mat to the shut-off valves.
- Open the shut-off valves.
- To change the mat, close the shut-off valves.

Step 3: Connect the device to the power supply and press the power button



- The „beep“ signals that the power is switched on.
- During the initial setup, the error code „E1“ (water level too low) is displayed.
- The water level indicator will flash accompanied with several „beeps“.

Step 4: Fill the device with distilled water

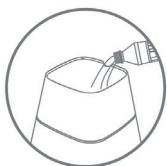

- Open the water inlet cover and fill with water until the water level indicator goes out.
- The water level indicator may flash again when the water flows into the heating mat.
- Keep adding water until the display turns off.
- Close the lid when the device and the heating mat are full.



Only use distilled water!

3. OPERATION

3.1 Setting the temperature

- Press the up and down arrow buttons to set the desired temperature.
- The heating temperature can be set between 10 °C and 45 °C.
- The current water temperature is shown on the device display. This value approaches the desired temperature when the device circulates the heated water.

3.2 Setting the timer mode

- There are 3 different modes available.
 - C1 – Normal timer mode**
 - C2 – Cooler timer mode**
 - C3 – Warmer timer mode**
- Press and hold the timer mode button for 2 seconds to tare between modes.
- To exit timer mode, press one of the arrow buttons for the temperature setting.

3.2.1 Normal mode (8 h)

- Maintains a temperature of 36 °C for the first 30 minutes.
- After 30 minutes, the temperature drops to 32 °C, which is maintained for 6 hours.
- In the last 1.5 hours, the temperature returns to 36 °C.

3.2.2 Cooler mode (14.5 h)

- Maintains a temperature of 32 °C for the first 30 minutes.
- After 30 minutes, the temperature drops to 30 °C, which is maintained for 6 hours.
- After a further 60 minutes, the temperature drops to 28 °C, where it is maintained for 4 hours.
- In the last 2.5 hours, the temperature rises back to 32 °C.

3.2.3 Warm mode (14 h)

- Maintains a temperature of 38 °C for the first 30 minutes.
- After 30 minutes, the temperature drops to 36 °C, which is maintained for 6 hours.
- After a further 60 minutes, the temperature drops to 34 °C, where it remains for 4 hours.
- In the last 2.5 hours, the temperature rises back to 38 °C.

3.3 Automatic dimming mode

- If no button is pressed for a period of 30 seconds, the light strip switches to dimming mode.
- To change the device control again, simply press any button on the device.

4. SPECIAL SETTINGS

4.1 Setting the timer

- The timer can be set from 1 to 24 hours. Once the preset time has elapsed, the device switches off automatically.
- After pressing the timer button twice, the display flashes.
- Set the desired time by pressing the up and down arrow buttons.
- As soon as the target time is displayed, the setting is active after 5 seconds and the display switches to the temperature display.
- Press the timer button twice or switch the device off / on to deactivate the timer function.
- The device does not switch off automatically if the timer is set to „00“.

4.2 Setting the childproof lock

- The childproof lock is used to prevent unwanted operation.
- With the device switched on, press the childproof lock button to switch the function on or off.
 - **Display E5** **LOCK ON**
 - **Display EC** **LOCK OFF**
- When lock mode is activated, the buttons are deactivated.

4.3 Switch between Fahrenheit (°F) and Celsius (°C)

- Press and hold the switch-on button for 20 seconds until „D1“ (°C) or „D0“ (°F) is displayed.
- Restarting the system activates the desired value.

5. SPARE PARTS

Art. No.	Description
E64802001	Warming Mat, small, L 55 x W 55 cm
E64802002	Warming Mat, large, L 55 x W 100 cm

For veterinary use only!

As with all warming devices patients must be checked regularly for pressure points and possible overheating.



GERMANY

EICKEMEYER KG
Eltastraße 8
78532 Tuttlingen
T +49 7461 96 580 0
F +49 7461 96 580 90
info@eickemeyer.de
www.eickemeyer.de

SWITZERLAND

EICKEMEYER AG
Sandgrube 29
9050 Appenzell
T +41 71 788 23 13
F +41 71 788 23 14
info@eickemeyer.ch
www.eickemeyer.ch

POLAND

EICKEMEYER Sp. z o.o.
Al. Jana Pawła II 27
00-867 Warszawa
T +48 22 185 55 76
F +48 22 185 59 40
info@eickemeyer.pl
www.eickemeyer.pl

ITALY

EICKEMEYER S.R.L.
Via G. Verdi 8
65015 Montesilvano (PE)
T +39 085 935 4078
F +39 085 935 9471
info@eickemeyer.it
www.eickemeyer.it

DENMARK

EICKEMEYER ApS
Solbakken 26, Hammelev
6500 Vojens
T +45 7020 5019
info@eickemeyer.dk
www.eickemeyer.dk

NETHERLANDS

EICKEMEYER B.V.
Doejenburg 203
4021 HR Maurik
T +31 345 58 9400
info@eickemeyer.nl
www.eickemeyer.nl

UNITED KINGDOM

EICKEMEYER Ltd.
3 Windmill Business Village
Brooklands Close
Sunbury-on-Thames
Surrey, TW16 7DY
T +44 20 8891 2007
info@eickemeyer.co.uk
www.eickemeyer.co.uk

CANADA

EICKEMEYER Inc.
617 Douro Street
Suite 205
Stratford, Ont. Canada
N5A 0B5
T +1 519 273 5558
info@eickemeyervet.ca
www.eickemeyercanada.ca