






### How to select different lens – Laser Therapy RLT-24

Five Lens	Principles of Application	Remarks
Zoom Optical(25mm) 	Non-contact type, for use by small and medium animals; For open wounds or exudate areas, or very painful and sensitive areas; Usually scanning treatment; The spot size is adjustable, the larger the number, the larger the spot, and the lower the power density under the same tissue distance;	The vertical distance to the treatment surface is usually 1~2cm
Massage Optical (25mm) 	Contact type, for use by small and medium animals; Usually used on light-colored hair, short hair or shaved animals; Point-to-point or Scanning;	
Plus Optical (50mm) 	Non-contact type, for use by large animals; For open wounds or exudate areas, or very painful and sensitive areas; Usually scanning treatment;	The vertical distance to the treatment surface is usually 1~2cm
Massage Plus Optical (50mm) 	Contact type, for use by large animals; Usually used on light-colored hair, short hair or shaved animals; Point-to-point or Scanning;	
ENT acupuncture handpiece 	Laser acupuncture program; For the treatment of ear, nose and throat of small animals, the power is limited to less than 3W;	

### Simple Guide on laser treatment for different diseases

Disease	Therapeutic schedule	notes
Stomatitis	One time per day for 2 times, than one time every other day for 4 times	It's better to scaling tooth first , then start the laser therapeutic schedule
Surgical incisions	One time per day for 3 times	
Arthritis	Two times every week for 12 times	
Edema / Swelling	One time per day for 3 times	If necessary it can be extended, one time every other day for 3 times
Tooth extractions	One time immediately after procedure	If necessary it can be extended, one time per day for 3 times
Fracture	One time per day for 3-6 times, than one time every two day for 5-7 times	If necessary it can be extended, two times every week for 6 times
Wounds	One time per day for 6 times	
Surgical incision pain	One time immediately after procedure	For continued pain, a second treatment may be given the next day
Chronic pain	Three or four times every week, then one time every 4-6weeks	Most chronic cases need continuous maintenance therapy